

Use
with Any
Disposable Brief

The Home-Caring Brand®
TRANQUILITY
Incontinence Products®

TopLiner™ Booster Pads and Contour

Adds Absorbency / Stops Leakage / Reduces Cost

All **TopLiner Booster Pads** line the inside of any disposable brief or undergarment. The flow-through design allows the pad to fill to capacity and then pass additional fluid to disposable undergarment boosting absorbency, maintaining skin integrity and controlling odor.



- Place pad on target area to control leakage and eliminate wetness to linen and clothing
- Higher absorbency reduces need for nighttime changes and allows for uninterrupted sleep
- Saves money by extending use of primary undergarment
- Latex-free

TopLiner Booster Contour Pads are a perfect solution for bowel incontinence.

- Hourglass shape provides larger absorbent area
- Soft, absorbent texture makes cleanup easier
- Easy to remove and replace with clean pad
- Latex-free

How to Position TopLiner Booster Contour Pad in Disposable Underwear

1. In seated position, put on the disposable underwear and pull it up to the knees.
2. Fold the wings of the Booster Contour under and place between the inner leg cuffs, making sure the pad is completely inside the inner leg cuffs.
3. Pull up the disposable underwear for a snug fit.

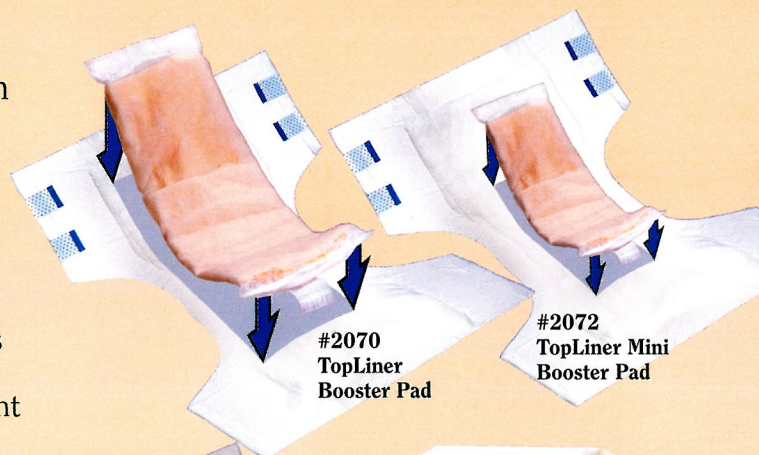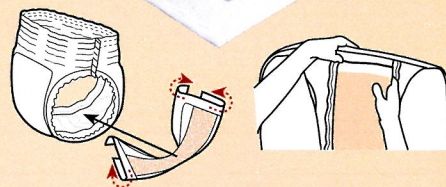

Item #	Description	Size	Capacity	Packaging
2072	TopLiner Mini Booster Pad	10.5" x 2.75"	5.7 oz /169 ml	8 - 25's
2070	TopLiner Booster Pad	14" x 4"	10.9 oz /322 ml	8 - 25's
3096	TopLiner Booster Contour	21.5" x 14"	13.6 oz /403 ml	10 - 12's

Principle Business Enterprises, Inc.
P.O. Box 129 • Pine Lake Industrial Park
Dunbridge, OH 43414-0129
Phone: 419-352-1551 • Fax: 419-352-8340
www.tranquilityproducts.com • consumerinfo@pbenet.com

Free Samples and Consultation
Call Tranquility's Care Center
1-800-467-3224